ROLLY

890.01

CE

Regulation (EU)
2017/745

EN

WWW.KONG.IT

Download the digital version

TABLE OF CONTENTS

1 - SYMBOLS AND SUPPORT	4
1.1 Symbols	4
1.2 Support	4
2 - GENERAL INFORMATION	5
3 - TECHNICAL FEATURES	6
3.1 Terminology and materials of the parts	6
3.2 Technical data	8
3.3 Optional components and spare parts	9
4 - SPECIFIC INFORMATION	10
4.1 Intended use	10
4.2 Preparation	10
4.3 Setup	11
4.4 Blocking the patient	16
4.5 Transporting the patient	18
5 - MAINTENANCE AND REPAIRS	20
5.1 General	20
5.2 Maintenance	20
5.3 Repair	20
6 - STORAGE	21
7 - CHECKS, INSPECTIONS AND SERVICING	22
7.1 Checks	22
7.2 Inspections	22
7.3 Servicing	22
8 - PRODUCT LIFE AND GUARANTEE	23
8.1 Product life	23
8.2 Disposal	23
8.3 Guarantee	23
8.4 Law obligations	23
9 - LABELLING AND SYMBOLS	24
9.1 Medical device label	24
9.2 Symbols	24
10 - DOCUMENTS	25
10.1 Inspections and servicing register	25
10.2 Maintenance and repair register	26
10.3 EU Declaration of conformity	27
11 - LIST OF APPLIED STANDARDS AND REFERENCE	28
11.1 Applied standards	28
11.2 Standards used as reference	29

1.1 SYMBOLS

To make reading this manual comfortable and clear, the following are the symbols used to manage important warnings, for correct and safe use of the device.

REQUISITE FOR CORRECT USE

It identifies the presence of information required to use the device correctly.

INFORMATIVE REQUISITE

It identifies the presence of useful and general information, to guide the user to consciously use the device and/or carry out the actions.

It identifies that the Medical Device is manufactured, designed and produced in compliance with the provisions of the General Safety and Performance Requirements of Regulation (EU) 2017/745 (Class I medical device, in compliance with classification rule 1 as indicated in the attachment VIII).

1.2 SUPPORT

For information, contact Kong Customer Service by:

- telephone +39 0341 630506,
- fax +39 0341 641550,
- email: safetycare@kong.it,

or write to KONG S.p.A. – Via XXV Aprile, 4 – 23804 Monte Marenzo LC - ITALY.

To facilitate assistance operations, always communicate or indicate the serial number (SN) indicated on the label applied to the Medical Device.

CHAPTER 2

2 GENERAL INFORMATION

The information provided by the manufacturer (hereinafter referred to as information) must be read and clearly understood by the user prior to using the Medical Device (MD hereafter). The information regards the description of the features, performance, assembly, disassembly, maintenance, preservation, disinfection, etc. of the device. Even though the information offers tips on use, this information shall not be considered as a user manual under actual conditions of use.

WARNINGS AND LIMITATIONS OF USE:

- this device shall be strictly used by people who are physically fit, trained (instructed and taught) to use the device and with specific experience regarding moving the patient or, during training activities, by people under direct supervision of the trainers/ supervisors who guarantee the safety thereof,
- do not use the device before fully reading and understanding this user manual,
- strictly follow the manufacturer's information, improper use of the device is hazardous,
- modifying and/or repairing the device is strictly forbidden,
- all checks described in chapter 7 must be carried out prior to and after using the device. In case of any doubt on the efficiency of the device, the user must replace it immediately,
- non-compliant use, deformations, falls, wear, chemical contamination, exposing textile/plastic components/devices to temperature below -30°C or above $+50^{\circ}\text{C}$ and metal components/devices to

temperatures exceeding 100°C are some examples of causes that can reduce, limit and end the life of the device,

- Prior to any rescue operation, be keen not to exceed the load capacity indicated in paragraph 3.2.2,
- in order to reduce risks of exposure to / transmission of infectious diseases, clean and disinfect the device as indicated in chapter 5,
- improper use of the patient immobilisation systems can jeopardise the safety of the patient,
- always check the compatibility of the devices used alongside the device by consulting the relative manufacturer's information,
- use of spare parts or optional components different from the ones indicated in paragraph 3.3 can be hazardous,
- do not expose the device to sources of heat and at contact with chemical substances. Reduce exposure to direct sunlight as much as possible. At low temperatures and in the presence of humidity, formation of ice could reduce flexibility and increase cutting and abrasion-related risks on textile and synthetic devices.
- Report to the manufacturer and the competent authority of the member state where the user and / or patient is established any serious incident occurring in relation to the MD.

All our devices are tested/ checked piece by piece in compliance with the procedures laid down by the Quality System certified in accordance with the UNI EN ISO 9001 standard. Laboratory tests, testing, information and standards do not always reproduce the practical result. Thus, the results obtained under the actual conditions of use of the device may differ, even considerably at times. The best information lies in the continuous practical use under the supervision of skilled/expert/qualified people.

Warning: not suitable for use in an ATEX environment (Directive 94/9/CE)

CHAPTER 3

TECHNICAL FEATURES

3.1 TERMINOLOGY AND MATERIALS OF THE PARTS

- A - Support surface in high-density polyethylene
- B1 - Slings in polyester
- B2 - Slings in polyester
- C - Slings in polyester with eyelets
- D - Slings in polyester
- E - Sling in polyester
- F - Handles in polyester
- G - Hanging kit in high-tenacity polyester (G1 short part - G2 long part)
- H - Static rope
- I - Buckle with hook in aluminium alloy
- L - Buckle with hook in aluminium alloy
- M - Slings with eyelets
- N - Sling with ratchet
- O - Transport bag.

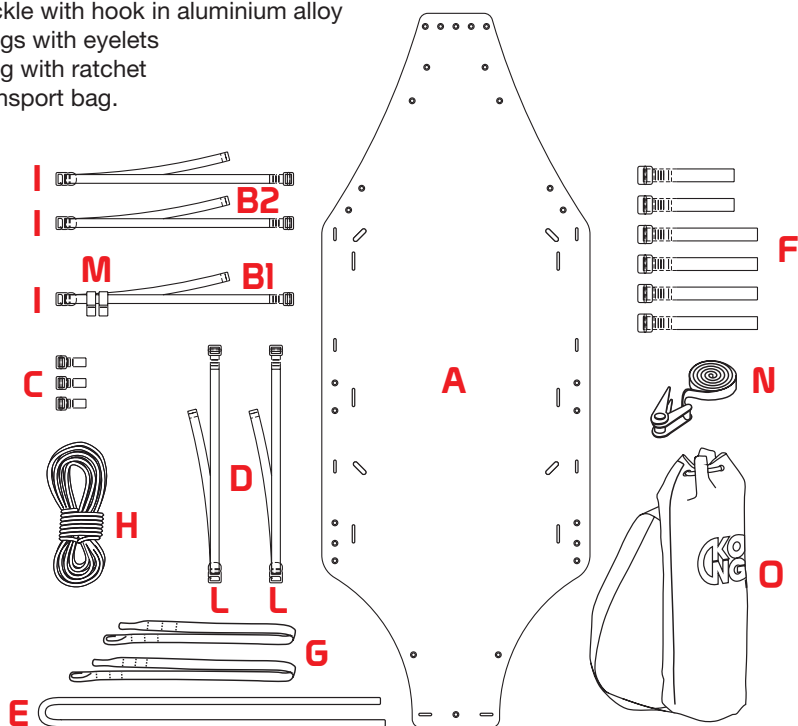
Fig.1

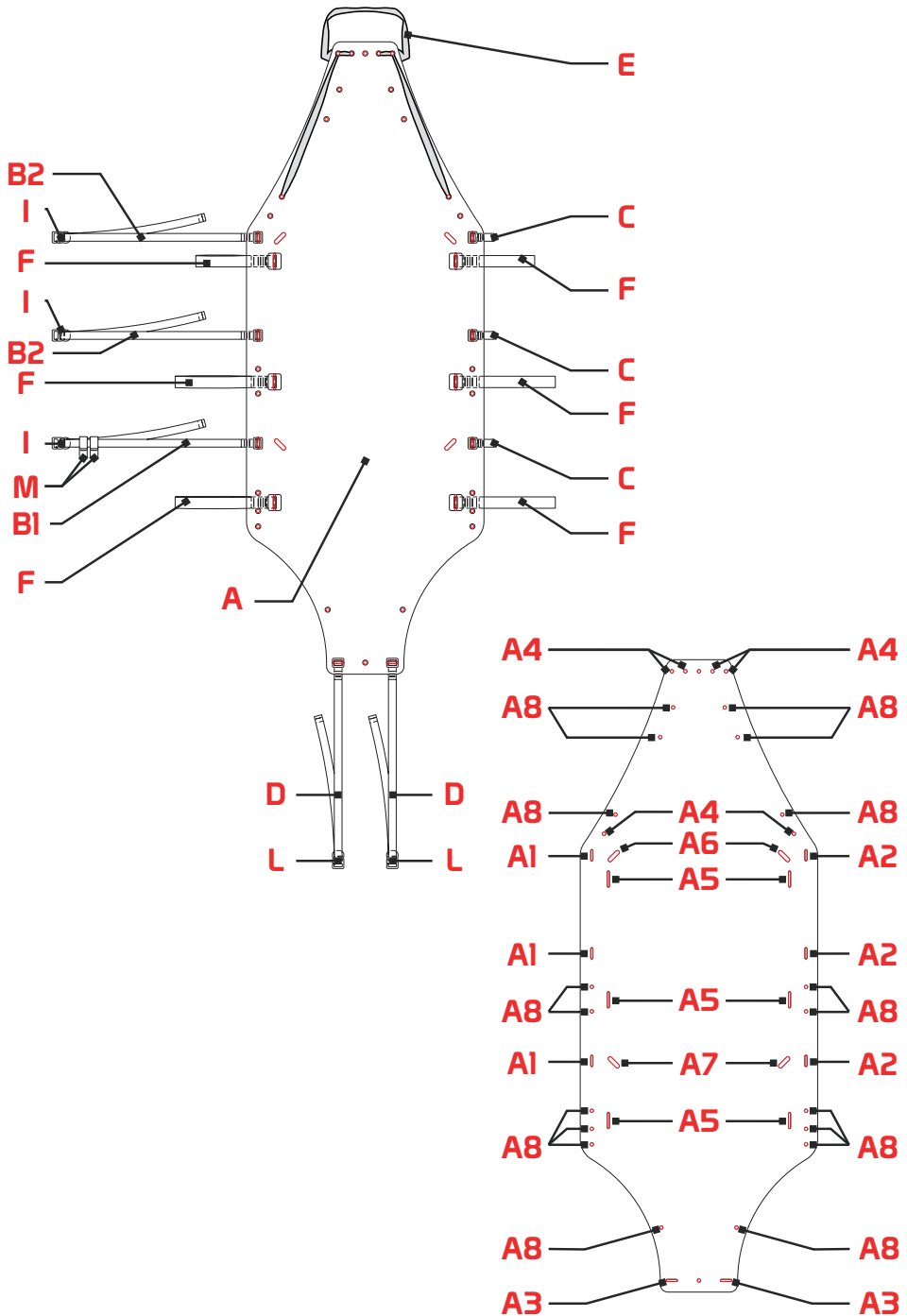
Fig.2

3.2 TECHNICAL DATA

3.2.1 Weight and dimensions

Length:	245 cm
Width:	92 cm
Rolled in the bag:	Ø 30 x 110 cm
Overell weight of the parts (fig. 1)	9 kg

3.2.2 Load capacity

Mode of use	Load capacity
Manual transport with the handles (F) (fig. 16):	250 kg
Lifting and lowering with rescue devices (fig. 17) with the hanging kit (G)	2100 kg
Vertical descent and recovery (fig 18) with the rope (H)	2400 kg
Hoisting and lowering with helicopters (fig. 17) with the hanging kit (G)	2100 kg

3.2.3 Buoyancy force

Mode of use	Buoyancy force
River transportation (fig. 4) with kit 880020300KK FLOK-IT	400 N

The safety factor shall be known and defined by the user.

3.3 OPTIONAL COMPONENTS AND SPARE PARTS

The methods of connection and use of the optional components are defined in the relative user instructions.

3.3.1 Optional components

880020300KK FLOK IT

Floating kit for stretchers, made of three cylinders of different lengths (2 side pieces of 1 m each and 1 piece head side of 0,4 m), each of one equipped with two fastening straps with quick release plastick buckles.

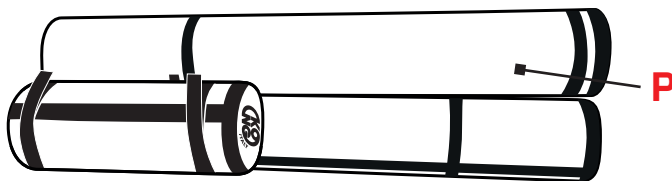
Fig.3

3.3.2 Spare Parts

- B1 - Slings in polyester
- B2 - Slings in polyester
- C - Slings in polyester with eyelets
- D - Slings in polyester
- E - Sling in polyester
- F - Handles in polyester
- G - Hanging kit in polyester high tenacity (G1 short part - G2 long part)
- H - Static rope
- N - Sling with ratchet
- O - Transport bag.

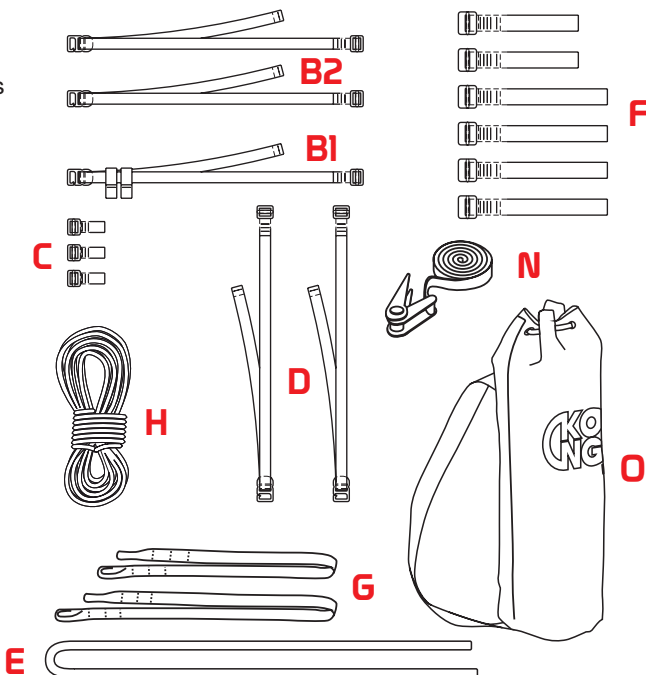
Fig.4

CHAPTER 4

SPECIFIC INFORMATION

4.1 INTENDED USE

The “**ROLLY**” rescue sheet is a MD that is particularly suited to recovering and transporting a patient - even immobilised on “**X-TRIM**” spinal boards or on the “**VACUUM**” mattresses, in a quick manner and in particularly difficult situations.

Decisions concerning moving and immobilising the patient, as well as the duration, method to be used and combination with other devices, shall be taken and executed by expert and trained personnel only.

The supplied hanging kit (G) makes it possible to use the “**ROLLY**” sheet with a winch.

4.2 PREPARATION

- Remove the “**ROLLY**” rescue sheet from the transport bag (O) and remove the ratchet sling (N)
- Distend the support surface (A) and flatten it, rolling the ends over in the opposite direction and folding the central section (fig. 5)

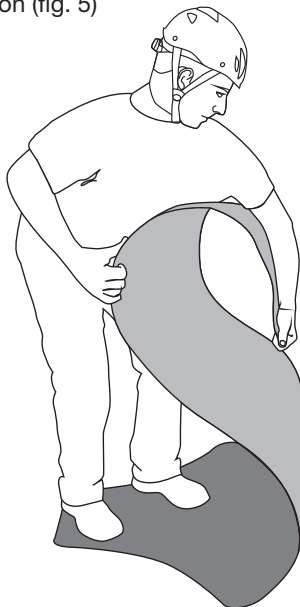
Fig. 5

4.3 SETUP

4.3.1 Setting up for blocking the patient

Prepare the following setup to block the patient:

- Insert the sling (E) in the A4 holes in the support surface (fig. 6).
- Insert the slings (B1 and B2) in the A1 eyelets in the support surface (fig. 7).
- Insert the slings with eyelets (C) in the A2 eyelets in the support surface (fig. 8).
- Insert the slings (D) in the A3 eyelets in the support surface (fig. 9).

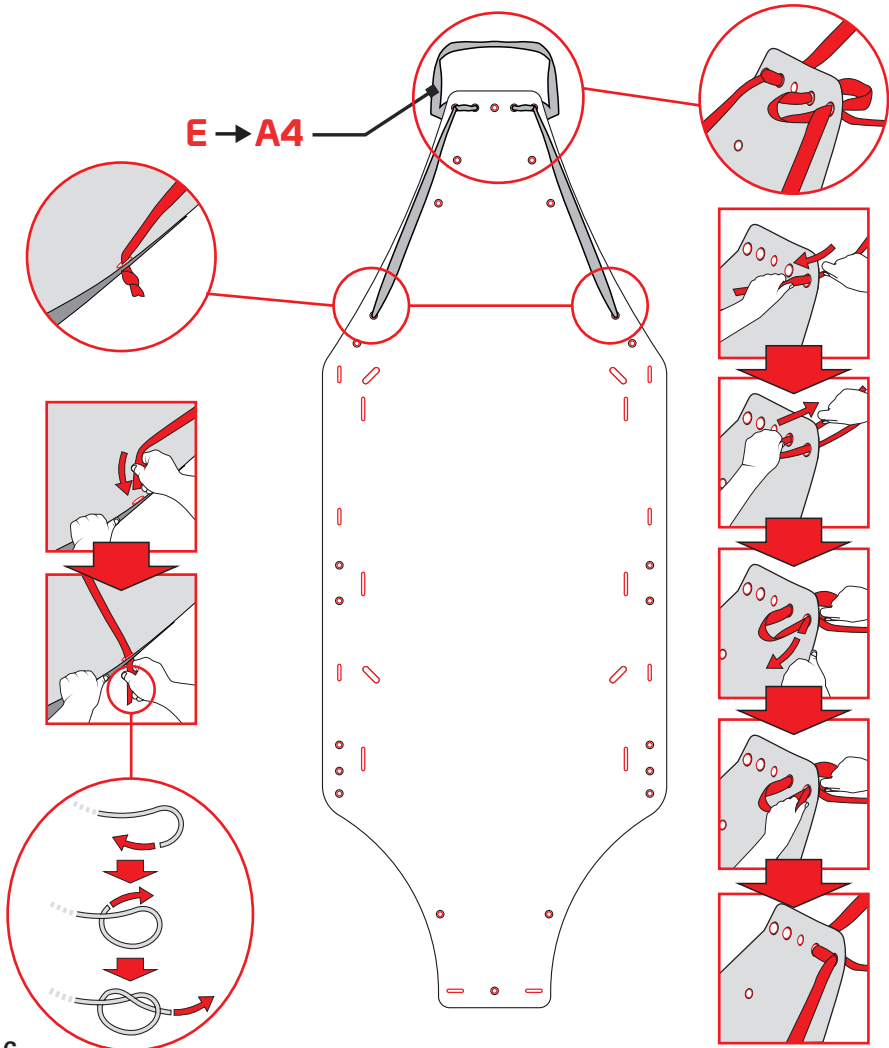
Fig.6

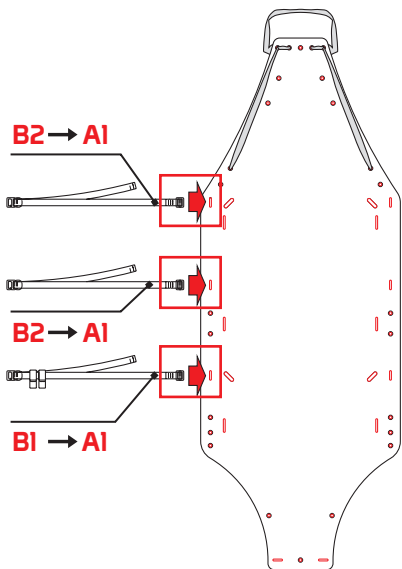
Fig.7

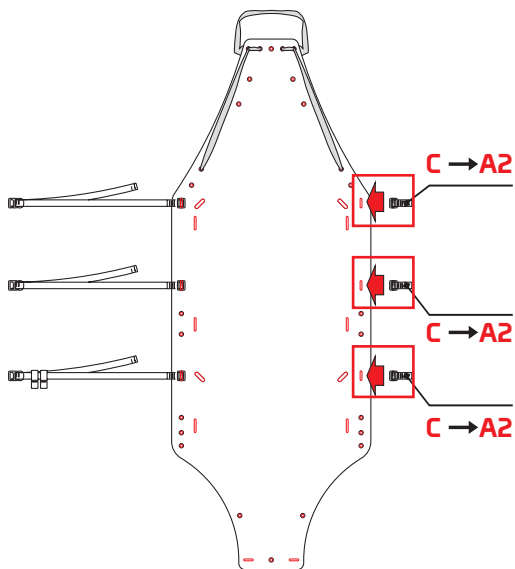
Fig.8


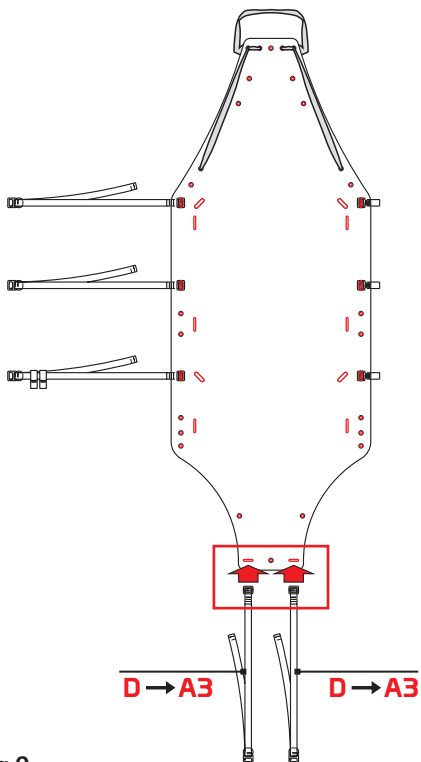
Fig.9

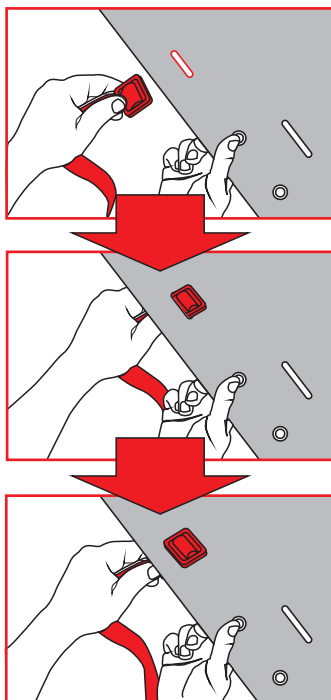4.3.2 Setting up for moving the patient

In relation to the lifting and transport procedures, which the rescue workers considers suitable, set the stretcher up with at least one of the following parts:

a) manual transport: insert only the side handles (F) in the eyelets A5 on the support surface (fig. 10) - Please note: we suggest inserting the shortest couple of handles in the place near the torso of the patient, in order to keep the head higher than the feet.

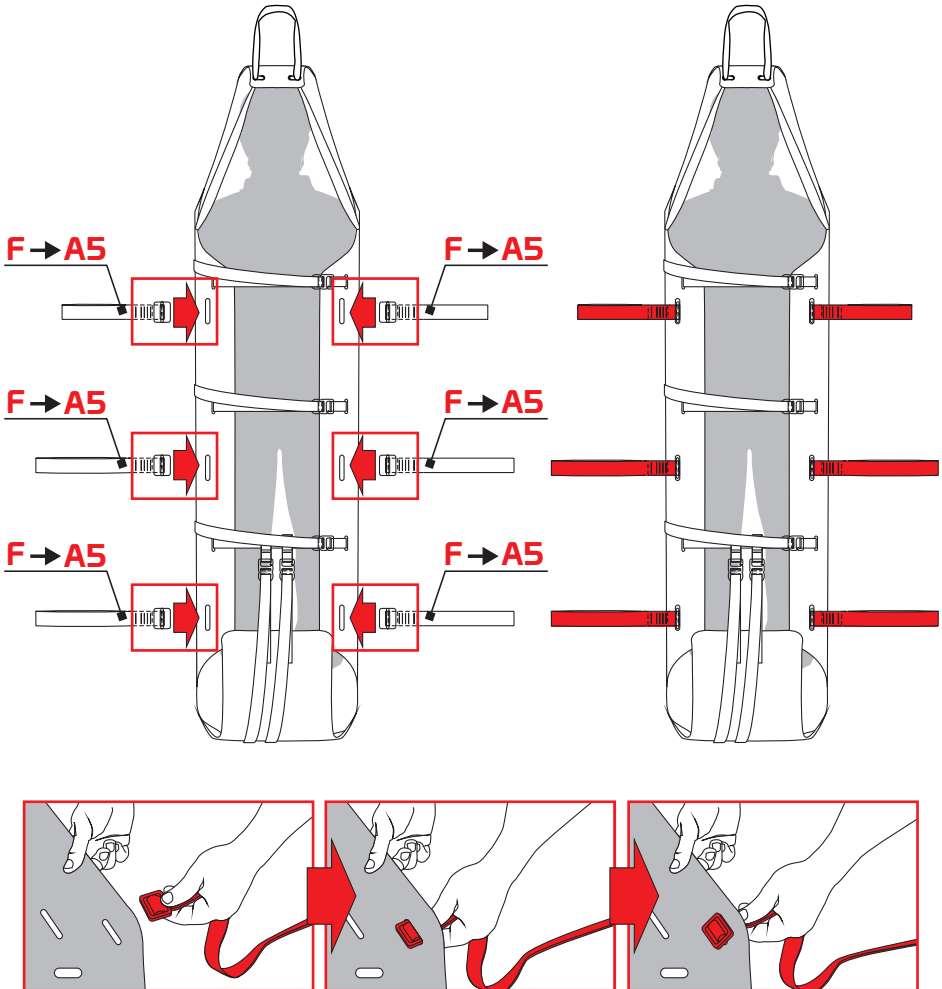
Fig.10

b) lifting/lowering and/or transportation using the lifting kit (G): insert the short hanging kit (G1) in the eyelets A6 and the long hanging kit (G2) in the eyelets A7 on the support surface (fig. 11).

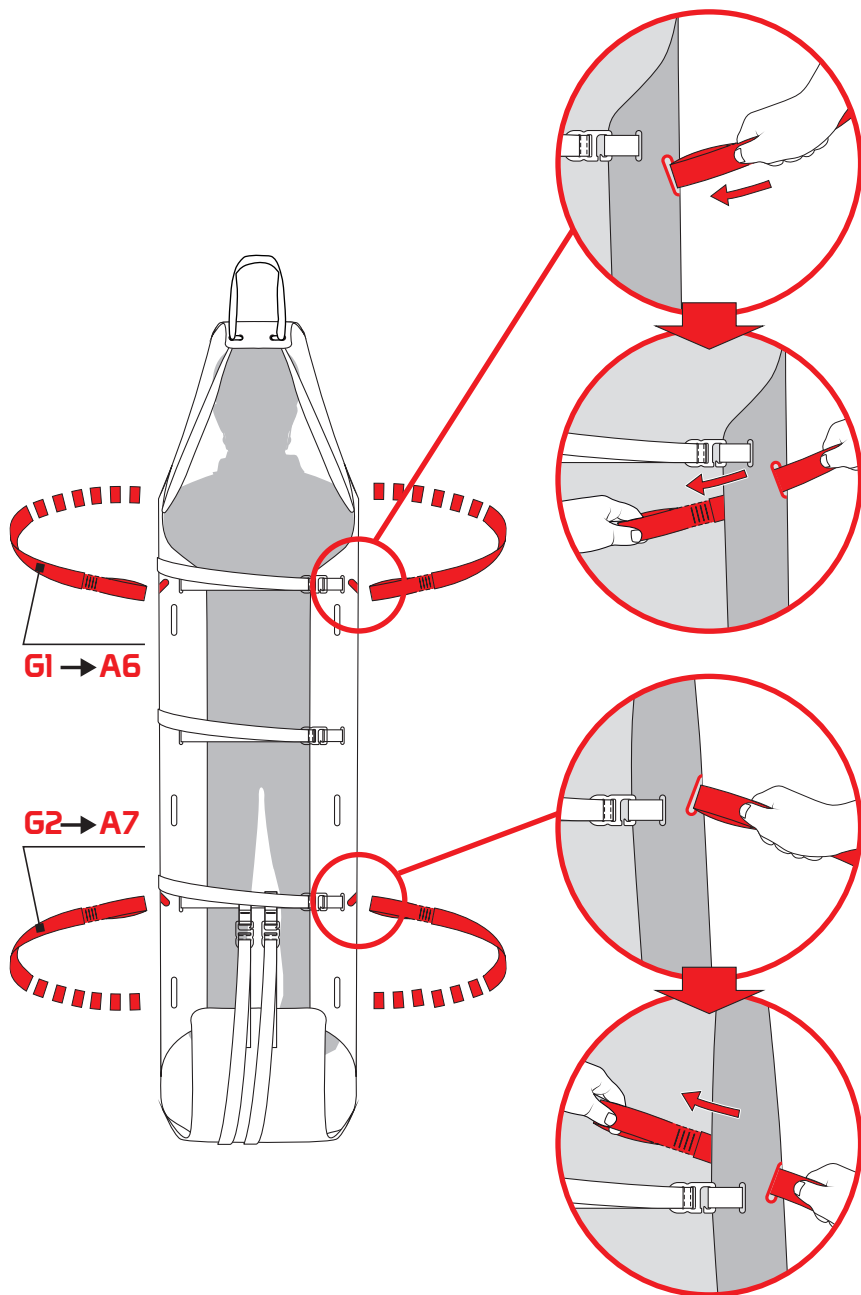
Fig.11

c) lifting/lowering or vertical lowering: insert the rope (H) in the holes A8 in the support surface (fig. 12).

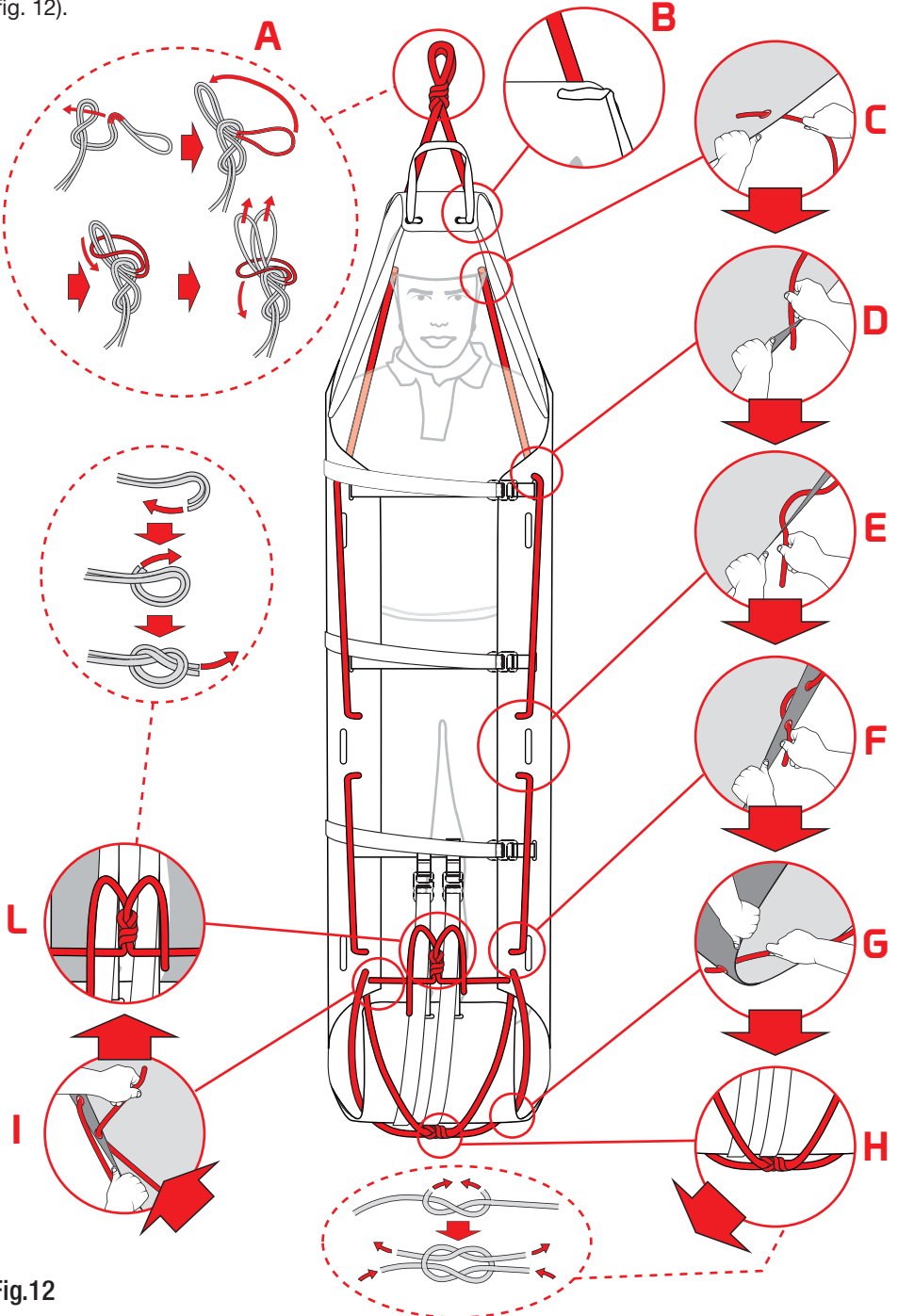
Fig.12

4.4 BLOCKING THE PATIENT

After having positioned the patient on the support surface (A):

- Insert the buckles with hook (I) in the corresponding slings with eyelet (C) and tension the slings (B) to lift the side parts of the support surface (A) enough to contain the patient (fig. 13).
- Insert the buckles with hook (L) in the corresponding slings with eyelet (M) and tension the slings (D) to lift the end part of the support surface (A) enough to contain the patient's feet (fig. 14).

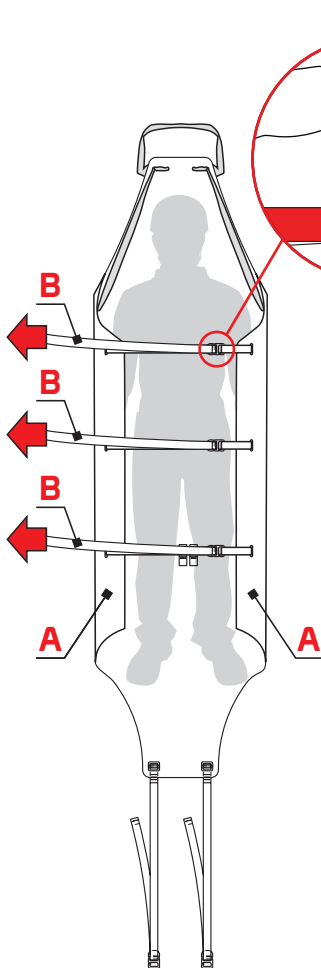
Fig.13

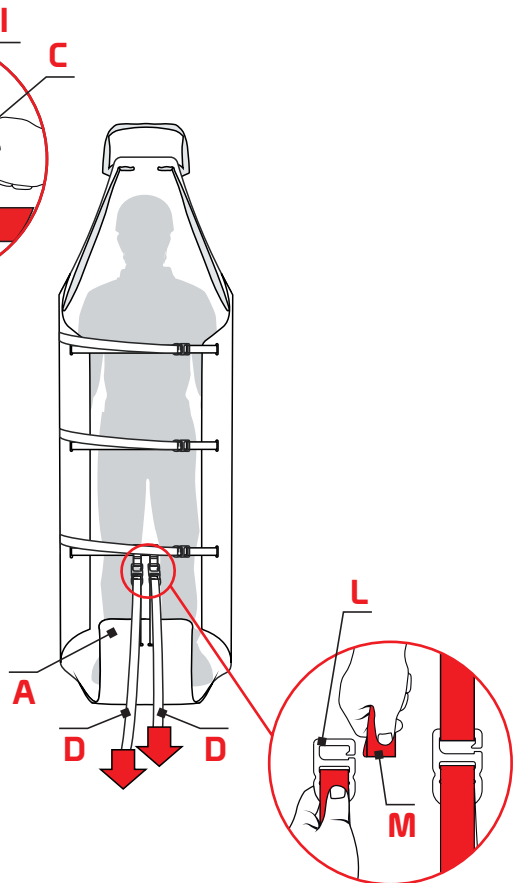
Fig.14

- c) Tension the sling (E) to lift the end part of the support surface (A) enough to contain the head of the patient and immobilise it (fig. 15).

Avoid prolonged contact of the patient's skin with the fabric of the device.

This procedure must also be carried out with patients immobilised on the spinal board "X-TRIM" or on the vacuum-operated mattress "VACUUM".

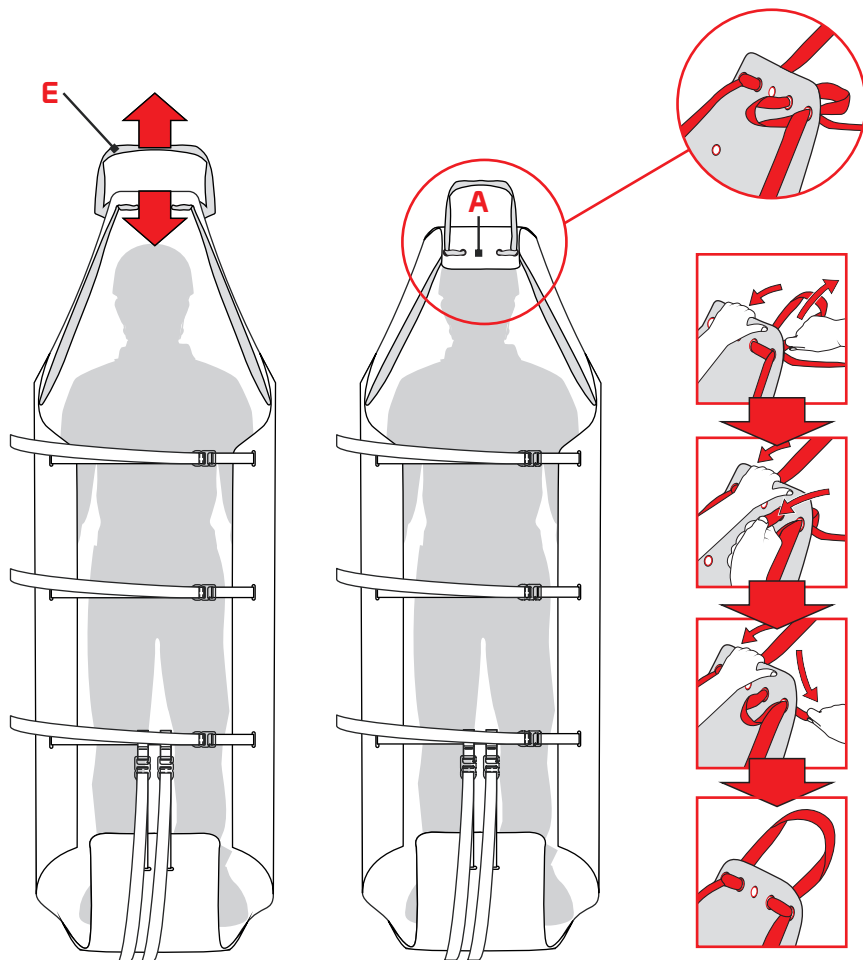
Fig.15

4.5 TRANSPORTING THE PATIENT

The “**ROLLY**” rescue sheet is suitable for use in the following modes of use:

- **Transportation with the handles (F) (fig. 16)**
- **Lifting and lowering with rescue devices (fig. 17)**
with the hanging kit (G)
- **Vertical descent and recovery (fig. 18)**
with the rope (H)
- **Hoisting and lowering with helicopters (fig. 17)**
with the hanging kit (G)
- **River transportation (fig. 19)**
with the 880020300KK FLOK-IT (L)

16 Before any recovery operation, make sure that the weight does not exceed the defined load capacity!

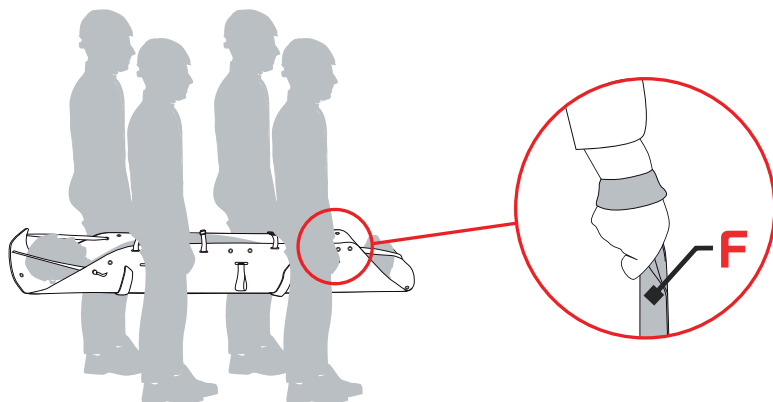
Fig.16

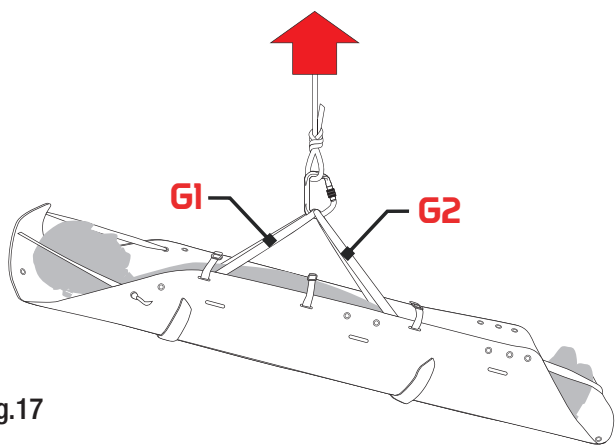
Fig.17

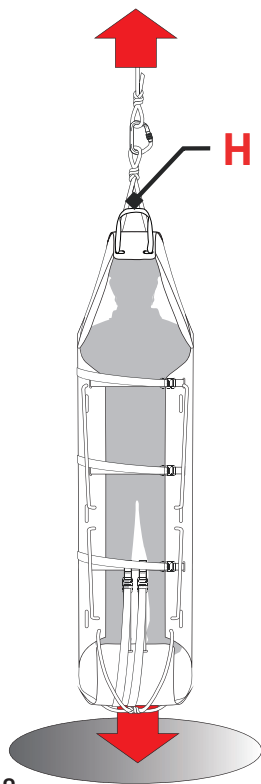
Fig.18

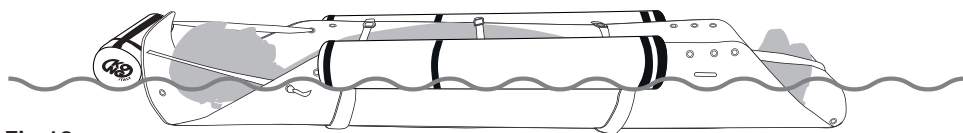
Fig.19

CHAPTER 5

MAINTENANCE AND REPAIR

5.1 GENERAL

The “**ROLLY**” rescue sheet is made of materials that are highly resistant to wear and to external agents. Despite this, the use conditions require maintenance operation and even repair in particular conditions.

Maintenance and repair operation must be recorded in a special register an example of which is indicated in chapter 10.

5.2 MAINTENANCE

The maintenance operations that must be carried out by the user are, in order:

- a) cleaning: wash the device by hand in lukewarm water (30-32°C) with neutral detergent (pH 5.5-8.5) and allow it to air dry, avoiding direct exposure to the sun and heat sources,
- b) disinfection: wipe with a clean cloth , soaked in a water solution of sodium hypochlorite (bleach) with a 0,1% concentration (1000 ppm). When blood or other bodily fluids are present, the recommended concentration of sodium hypochlorite is 0,5% (5000ppm)

5.3 REPAIR

Repair shall be strictly carried out by the manufacturer.

The user is allowed to replace the parts mentioned in paragraph 3.4.2 only, with new and original parts.

CHAPTER 6

STORAGE

6 STORAGE

After cleaning, any disinfection and drying operations, keep the product and its optional components in a place that is dry (relative humidity 40-90%), cool (temperature 5-40° C) and safe (avoid UV radiations), chemically neutral (absolutely avoid salty and/or acid environments), away from sharp edges, sources of heat, humidity, corrosive substances or other possible jeopardising conditions.

Do not store this device wet!

CHAPTER 7

CHECKS INSPECTIONS AND SERVICING

7.1 CHECKS

Before and after every use, check the device and ensure that:

- it is appropriate for the use it was intended for,
- it has not been subjected to mechanical deformations and it neither has cracks nor shows signs of wear,
- the textile parts do not have cuts, burns, chemical products residues, excessive hair, wear, in particular check areas at contact with the metal components,
- markings, including labels, are readable.

7.2 INSPECTIONS

The device must be inspected at least once a year, starting from the date of first use, by expert personnel trained and approved by KONG S.p.A. The date of the first use and the outcome of the inspection must be recorded in the Inspections and Servicing Register, regarding which an example is indicated in chapter 10.

7.3 SERVICING

The device must be serviced by KONG S.p.A., or by expert personnel trained and approved by KONG S.p.A. if:

- any malfunctions are detected,
- the result of the pre and post use check or of the inspections is negative.

The outcome of the servicing must be recorded in the Inspections and Servicing Register, regarding which an example is indicated in chapter 10.

The device subject of servicing has a one-year warranty from the date of servicing.

CHAPTER 8

PRODUCT LIFE AND GUARANTEE

8.1 PRODUCT LIFE

Carefully read the “Warnings and limitations of use” paragraph in chapter 2. The duration of this device depends on the outcome of the at least one-yearly inspection (paragraph 7.2) and possible servicing.

The hanging kit, even if not used, must be replaced after 10 years from the first use and in any case no later than 12 years from the date of manufacture.

8.2 DISPOSAL

Follow the rules in force in the country of use and the waste disposal procedures of the reference hospital facility for proper disposal.

8.3 GUARANTEE

The manufacturer guarantees the compliance of the device with the regulations in force at the time of production. Warranty covering defects is limited to defects regarding raw materials and manufacturing defects. The warranty does not cover normal wear, oxidation, damage caused by non-compliant use and/or in competition, by improper maintenance, transportation, preservation or storage. Warranty will be immediately deemed null and void should the device be modified or tampered with. Validity corresponds to the legal validity of the country where the device was sold, as from the date of sale, by the manufacturer. No claims shall be raised against the manufacturer once the aforementioned period has expired. Any repair or replacement request under warranty must be accompanied by a proof of purchase. Should the defect be recognised, the manufacturer undertakes to repair or, replace or refund the device at his discretion. The manufacturer shall not be held liable beyond the invoice price of the device in any case whatsoever.


8.4 LAW OBLIGATIONS

Professional and free time activities are often regulated by special national laws which can impose limits and/or obligations concerning the use of these devices. The user has the obligation to know and apply such laws which could provide for limits different from those indicated in this information.

CHAPTER 9

LABELLING AND SYMBOLS

9.1 MEDICAL DEVICE LABEL

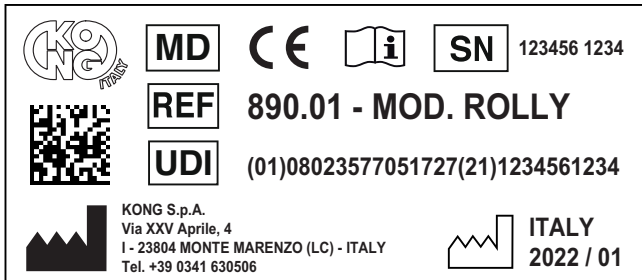Any changes to the positioning of the symbols does not change the content thereof.

9.2 SYMBOLS

Identification and traceability

	Medical Device
	Product identification code
	Unique Device Identifier
	Manufacturer's identification
	Unique serial number
	Country, year and month of production.

Data matrix UDI
 Readable with the Apps:
 iGepir - BarValid - Barcode Scanner

Information and safety

	Consult the user manual
	Compliant with Regulation (EU) 2017/745

CHAPTER 10

DOCUMENTS

10.1 INSPECTIONS AND SERVICING REGISTER

RESCUE SHEET - Class I MD				
REF.	890.01	MOD.	ROLLY	
UDI-DI	08023577051727	SN		
Date of first use				
INSPECTION AND SERVICING				
DATE	I/S	Description	Outcome	Supervisor

10.2 MAINTENANCE AND REPAIR REGISTER

RESCUE SHEET - Class I MD				
REF.	890.01		MOD.	ROLLY
UDI-DI	08023577051727		SN	
Date of first use				
MAINTENANCE AND REPAIRS				
DATE	M/R	Description	Outcome	Supervisor

10.3 EU DECLARATION OF CONFORMITY

The EU Declaration of Conformity is available for download at www.kong.it

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Declares under its sole responsibility that the device:

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V08050199 - Stretchers for the transfer of patients - others

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Intended use:

Stretcher for transporting patients during rescue operations

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www.kong.it

11.1 APPLIED STANDARDS

- EN ISO 14971: 2019
Medical devices - Application of risk management to medical devices.
- EN 62366-1: 2015 + A1: 2020
Medical devices - Application of usability engineering to medical devices.
- EN ISO 15223-1: 2021
Medical devices — Symbols to be used with information to be supplied by the manufacturer — Part 1: General requirements
- EN ISO 20417: 2021
Medical devices - Information to be supplied by the manufacturer
- EN ISO 10993-1: 2020
Biological evaluation of medical devices - Part 1: Evaluation and testing within a risk management process
- EN ISO 10993-5: 2009
Biological evaluation of medical devices - Part 5: Tests for in vitro cytotoxicity
- EN ISO 10993-23: 2021
Biological evaluation of medical devices - Part 23: Tests for irritation
- EN 597-1: 2015
Furniture - Assessment of the ignitability of mattresses and upholstered bed bases - Part 1: Ignition source smouldering cigarette
- EN ISO 846:2019
Plastics - Evaluation of the action of microorganisms
- EN 1865-1: 2010 + A1: 2015
Patient handling equipment used in road ambulances - Part 1: General stretcher systems and patient handling equipment

11.2 STANDARDS USED AS REFERENCE

- EN ISO 13485: 2016
Medical devices - Quality management systems - Requirements for regulatory purposes
- EN 13718-1: 2014 + A1: 2020
Medical vehicles and their equipment - Air ambulances - Part 1: Requirements for medical devices used in air ambulances.

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